Healbond Max 5 ml

MADE IN FRANCE

Adhésif universel

Universal bonding

Remarque

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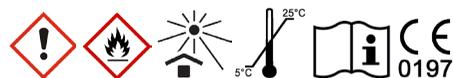
GB

Définition

Two-component adhesive system including a PRIMER and an ADHESIVE. The Primer provides effective etching to enamel and dentin without the need for an etching gel, thus simplifying the bonding procedure. The material’s chemistry allows for compatibility with all self-cure or dual-cure resin cements and core build-up materials. Dentists can therefore use HEALBOND MAX for direct and indirect procedures. The special low toxicity TEGDMA, BisGMA and HEMA-free formulation still ensures a strong and durable bonding. Depending on the case, this adhesive system can be used:

- In a "self-etching" mode, to enable the shortest possible treatment time and to minimize postoperative sensitivities.
- With selective enamel etching to maximize the adhesion to the enamel.
- With enamel and dentin etching ("total etch"), if the etching process cannot be restricted to the enamel, or when the tooth is devitalized.

Indications for use

Direct Bonding Application

- Light-cure, self-cure and dual-cure composite restorations.
- Composite and fractured ceramic/metal crowns repairs.
- Sealing of hypersensitive and/or exposed root surfaces.

Indirect Bonding Applications

- Veneers
- Indirect Inlay cores, and in conjunction with self-cure, dual-cure, and light-cure resin cements).
- Zirconia, ceramic (lithium disilicate-based and alumina-based), composite, and metal-based
- precious and non-precious) inlays, onlays, crowns, bridges.
- Endodontic posts.

Note

HEALBOND MAX is compatible with all composites, self-cure, dual-cure and light-cure resin-cements and core build-up materials, and it does not require the use of any dual cure Activator if it is used with our range of product.

Safety notes

Warnings

- The product contains methacrylate resins and alcohol which may irritate skin, eyes and oral mucosa. It may cause damage to the pulp and allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.
- The product is a corrosive liquid and is flammable. Keep away from sources of ignition.

Precautions

- The use of this product is restricted to dental professionals.
- The product must not be used for any purpose other than specified in the indications.
- Wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients

Contraindications

Product is contraindicated in patients with a history of allergic reactions to methacrylate resins.

Composition

PRIMER: silane - Phosphoric acid monomer (MDP) - Hydrophilic dimethacrylate - Camphorquinone (CQ) - ethanol and water.

ADHESIVE: urethane dimethacrylate (UDMA) - Phosphoric acid monomer (MDP) - hydrophilic dimethacrylate - Camphorquinone (CQ) - accelerators - Stabilizers - Nanofillers

Clinical procedures

Use one applicator tip for HEALBOND MAX Primer and another one for HEALBOND MAX Adhesive.

DIRECT RESTORATIONS

Bonding Composites to dentin/enamel

- Rubber dam is the recommended method if isolation to keep the working area free of blood and saliva, and to protect soft tissues.
- Prepare the cavity with minimal tooth reduction. Cavity floor very close to pulp should be covered with a thin layer of hard setting calcium hydroxide material or with an anti-microbial liner (e.g. Healdent).
- Rinse thoroughly with water spray and gently air dry (do not over-dry). The tooth structures should be just dry enough that the surface has a slightly glossy appearance.
- Apply HEALBOND MAX Primer onto the enamel/dentin surface for 20 seconds with a rubbing motion, using a disposable applicator brush.
- Air thin with medium air pressure until the solvent has evaporated and the primer does not move.
- Do not light cure.
- Shake HEALBOND MAX Adhesive bottle briefly and apply to the enamel/dentin surface.
- Slightly air thin to avoid pooling and to disperse the adhesive homogeneously.
- Light-cure for 10 seconds.

- Apply the composite of your choice per manufacturer’s instructions for use.

Note

HEALBOND MAX produces high enamel bond strengths without the use of etching. However, etching gel can be used on enamel and/or dentin if a total etch or selective enamel etch technique is desired. In that case, after step 2, etch Enamel or Dentin surfaces for 15 seconds, then rinse thoroughly with water spray for 15 seconds and gently air dry (do not over-dry). Proceed to step 3.

INDIRECT RESTORATIONS

Tooth structures preparation

- Prepare the abutment tooth in a conventional manner.
- Try in restoration to verify its good adaptation, especially at the limits of the preparation.
- Clean the tooth structures, rinse thoroughly with a water spray, and gently air dry. Do not over dry. The tooth structures should be just dry enough that the surface has a slightly glossy appearance.
- Apply HEALBOND MAX Primer onto the enamel/dentin surface for 20 seconds with a rubbing motion, using a disposable applicator brush.
- Air thin with medium air pressure until the solvent has evaporated and the primer does not move.
- Do not light cure.
- Shake HEALBOND MAX Adhesive bottle briefly and apply to the enamel/dentin surface.
- Slightly air thin to avoid pooling and to disperse the adhesive homogeneously.
- Light cure for 10 seconds.

Note

Light-curing of HEALBOND MAX Adhesive is optional if INNOCEM resin-cement is used for cementation. HEALBOND MAX produces high enamel bond strengths without the use of etching. However, etching gel can be used on enamel and/or dentin, especially in case of veneers with poor retention, and Maryland bridges. In that case, after step 1,3, etch Enamel or Dentin surfaces for 15 seconds, then rinse thoroughly ensuring that all etchant has been removed. Proceed to step 1.4.

Preparation of restorations

Treatment of Lithium Disilicate, Ceramic/Porcelain Restorations [etchable]

- Abrase the internal surface of the restorations with 50µm alumina with a pressure of about 30 psi [0.2 MPa].
- Etch the restoration with hydrofluoric acid for 1 minute.
- Rinse thoroughly with water for 15 seconds and dry with air free of water and oil.
- Apply HEALBOND MAX Primer on the internal surface of the restoration for 20 seconds with a rubbing motion, using a disposable applicator brush.
- Air thin with medium air pressure until the solvent has evaporated and the primer does not move.
- Do not light cure.
- Shake HEALBOND MAX Adhesive bottle briefly and apply to the internal surface of the restoration.
- Slightly air thin to avoid pooling and to disperse the adhesive homogeneously.
- Light cure for 10 seconds.

Note

HEALBOND MAX Primer can be used as silane replacement.

Light-curing of HEALBOND MAX Adhesive is optional if INNOCEM resin-cement is used for cementation.

Treatment of Metal-based, Zirconia-based, Alumina-based and Composite Restorations [non-etchable]

- Abrase the internal surface of the restorations with 50µm alumina with a pressure of about 60 psi [0.4 MPa] for metal-based, zirconia-based and alumina-based restorations, or a pressure of about 15 psi [0.1 MPa] for composite-based restorations.
- Clean the prepared surfaces with alcohol and dry with a water-free and oil-free air.
- Apply HEALBOND MAX Primer on the internal surface of the restoration for 20 seconds with a rubbing motion, using a disposable applicator brush.
- Air thin with medium air pressure until the solvent has evaporated and the primer does not move.
- Do not light cure.
- Shake HEALBOND MAX Adhesive bottle briefly and apply to the internal surface of the restoration.
- Slightly air thin to avoid pooling and to disperse the adhesive homogeneously
- Light cure for 10 seconds.

Note

Light-curing of the material is optional if INNOCEM resin-cement is used for cementation.

Cementation and seating of restoration

Veneers, Crowns, bridges, inlays, onlays and metal-based restorations

- Rubber dam is the recommended method if isolation to keep the working area free of blood and saliva, and to protect soft tissues.
- Apply INNOCEM (or another resin cement of your choice) to the restoration and seat it.
- Cure the cement per manufacturer’s instructions for use.

Note

Ceramic restorations must be cemented before checking occlusion (risk of breakage)

Definitive cementation of root canal posts and bonding of dual-cure core build up composite

Preparation of root canals and posts

- Rubber dam is the recommended method if isolation to keep the working area free of blood and saliva, and to protect soft tissues.
- Clean the prepared root canal with a 5% solution of sodium hypochlorite (NaOCl).
- Rinse thoroughly with water spray. Dry with paper tips.
- Trial fit the post. It should fill 2/3 of the root canal.
- Clean the post with alcohol and dry it with water-free and oil-free air.

- Apply HEALBOND MAX Primer in the root canal preparation and on all the cavity walls involved in the bonding procedure for 20 seconds with a rubbing motion, using a disposable applicator brush.
- Air thin with medium air pressure until the solvent has evaporated and the primer does not move.
- Do not light cure.
- Shake HEALBOND MAX Adhesive bottle briefly and apply in the root canal preparation and on all the post surfaces.
- Slightly air thin to avoid pooling and to disperse the adhesive homogeneously.
- Apply a resin cement of your choice or a dual cure core-build up material onto the post, seat the post, vibrate the post slightly avoiding the possibility of trapped air, and then cure the resin cement or the dual cure composite per the manufacturer’s instructions.
- Proceed with a core build-up material of your choice in accordance with manufacturer’s instructions.

Note

Light-curing of the material is optional if INNOCEM resin cement is used for cementation.

Excesses of HEALBOND MAX Adhesive can be removed with a dry applicator brush or an absorbent paper point.

Storage & shelf-life

- The product must be used by the expiration date indicated on the package.
- The product should be tightly closed immediately after use.
- Store in a fridge between 2°C and 8°C when not in use. Bring it to room temperature, ½ hour before use.
- Never store the product close to sparkles and open fire.
- Do not expose the product to sun-light or warm sources.

Important notice

- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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