Definition
Desensitizer & liner.
Safety notes
Warnings

- Contains polymerizable methacrylate resins which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.

Precautions

- The use of this product is restricted to dental professionals.
- The product must not be used for any purpose other than specified in the indications.
- Wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients.

Contraindication

Product is contraindicated in patients with a history of allergic reaction to methacrylate resins.

Composition

Hydrophilic methacrylate resin - water - benzalkonium chloride

Clinical procedures
Liner

1. Prepares cavities. If the cavity is deep, apply a calcium hydroxide layer before using HEALDENT.
2. It is not necessary to have a dry dentin surface before applying HEALDENT : a slightly wet surface is acceptable.
3. After peripheral enamel and dentin are etched, apply HEALDENT on the dentin with a disposable applicator, using gentle but firm rubbing motion. Do not rinse.
4. Wait 15 seconds and gently blow dry air.
5. Apply a bonding per Manufacturer's instructions and light-cure it. (HEALDENT is compatible with all bonding systems).
6. Apply a composite per Manufacturer's instructions and light-cure it.

Desensitizer
Technique 1

1. Providing the tooth is not too sensitive, clean the cervical area with pumice and rinse the area with plenty of water.
2. Apply HEALDENT for 30 seconds with a disposable applicator, using gentle but firm rubbing motion.
3. Lightly blot dry or let air dry. Do not rinse. Repeat application if required.

Technique 2

1. If area is too sensitive to clean with pumice:
2. Apply directly HEALDENT following technique 1.
3. Lightly blot dry or let air dry. Do not rinse.
4. Apply a second layer of HEALDENT.
5. If the patient is not hyper-sensitive anymore, apply a 3rd layer of HEALDENT. If patient is still hyper-sensitive, deliver a local anaesthetic and apply HEALDENT using Technique 1.

Other indications

1. After bleaching & whitening sessions.
2. After scaling, root planning & perio treatments.
3. In extreme cases where cervical sensitivity is a continuous problem, HEALDENT can be used under any definitive resin-based definitive. system.

Storage & shelf-life

- The product must be used by the expiration date indicated on the packaging.
- Do not expose the product to sun-light or warm sources.
- Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it

- should be brought back to room temperature 15 minutes minimum before use, in order to restore its normal viscosity and curing time.
- Use the product at room temperature.

Important notice

- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it ultimately unusable.

Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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