Gel astringent - 25% Sulfate d'aluminium  
4 x 1,2ml seringues + 10 embouts applicateurs

Astringent gel - 25% Aluminium sulfate  
4 x 1,2ml syringes + 10 dispensing tips

G-PHARMA | B.P. 10279 HERBLAY  
95617 CERGY-PONTOISE CEDEX FRANCE  
elsodent@elsodent.com | www.elsodent.com

## Hemosal

GB

### Definition

Astringent gel - 25% aluminum sulfate. Available in pre-filled syringe, in a gel form hygienic and ergonomic.

### Safety notes

#### Warnings

- Contains Aluminium Sulfate which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.

#### Precautions

- The use of this product is restricted to dental professionals.
- The product must not be used for any purpose other than specified in the indications.
- Wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients.

### Contraindication

Product is contraindicated if the patient is known to be allergic to Aluminium Sulfate.

### Composition

Water - aluminium salt - amorphous silica - flavours

### Clinical procedures

Put a disposable applicator on the syringe.

### Impression

1. Gently apply a thin layer of gel directly to the sulcus area.
2. Select the appropriate cord (00, 0 or 01).
3. Pack the selected cord through the gel, into the sulcus. Wait 2 to 6 minutes.
4. Remove the cord, rinse and dry.
5. Take the impression as usual.

### Parodontology & dentistry

1. Apply Hemosal with pressure, directly on haemorrhaging area, with a bristle brush or a Elsobrush.
2. Wait for 2 to 4 minutes and rinse.

### Endodontic treatments

1. Apply a paper cone into the gel and apply the prepared cone in the haemorrhagic canal.
2. For 20 to 30 seconds. Reapply another paper cone if necessary.

### Storage & shelf-life

- The product must be used by the expiration date indicated on the package.
- Do not expose the product to sun-light or warm sources.
- Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it should be brought back to room temperature 15 minutes minimum before use, in order to restore its normal viscosity and curing time.
- Use the product at room temperature.

### Important notice

- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

### Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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