**Definition**

Dual-cure material for temporary fillings. The slightly elastic consistency assures an easy removal, even in case of undercuts, and finishing the cavity is not necessary. Thus the fitting of the temporary filling will be improved. It is available in dual barrel syringes for direct application into the cavity. Prorez contains fluoride and continuously releases fluoride ions.

**Indications**

All types of temporary fillings, but especially for inlays and onlays cavities.

**Safety notes**

**Warnings**

- Contains polymerizable methacrylate resins which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.

**Precautions**

- The use of this product is restricted to dental professionals.
- The product must not be used for any purpose other than specified in the indications.
- Wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients.

**Contraindication**

Product is contraindicated in patients with a history of allergic reaction to methacrylate resins.

**Composition**

Hydrophilic dimethacrylates - modified urethane dimethacrylates - barium aluminosilicate glass - amorphous silica - Camphorquinone (CQ) - accelerators - stabilizers - pigments

**Clinical procedures**

**First use of a new automix-syringe**

- Remove and discard the cap of the automix-syringe. Under no circumstances should the cap of the automix-syringe be used to reseal the automix-syringe as this could result in the inclusion of air bubbles, inducing a cross-contamination (base/catalyst) and leading to a premature polymerization of the product.
- Check whether both automix-syringe openings are free and remove obstructing material, if any, with a suitable instrument.
- Before installing a mixing tip, bleed the automix-syringe onto a mixing pad until the base and catalyst pastes are at the same level

**Application**

- Prepare the tooth cavity in a conventional manner for inlays. Rinse and dry the cavity.
- Apply Prorez into the cavity through the intra-oral tip and slightly contour with a suitable instrument (round condenser, spatula, etc.). Contour with slight excess.
- For use in the inlay and onlay technique, keep the approximate contact points free of resin

- with dental floss or a matrix before curing.
- After complete curing of the material, check occlusion. Excess can be removed with a bur.
- Silicone points are suitable for polishing.

**Removal of the temporary filling**

- Insert a rigid instrument into the temporary filling and extract it.

**Precautions**

Adhesion to dental resins or linings (e.g. light-curing materials) may occur. In such cases apply a suitable insulating agent (e.g. glycerine, vaseline). Avoid use of lining materials containing eugenol, since these could impede the setting of Prorez. The slight expansion of Prorez ensures a tight and waterproof seal of the temporary restoration.

**Storage & shelf-life**

- The product must be used by the expiration date indicated on the package.
- Do not expose the product to sun-light or warm sources.
- Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it should be brought back to room temperature 15 minutes minimum before use, in order to restore its normal viscosity and curing time.
- Use the product at room temperature.

**Important notice**

- Leave the used mixing tip in place on the automix-syringe until next use.
- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

**Disclaimer**

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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