# Seal Temp S

## Definition

Resin-based temporary luting cement. Well-balanced adhesive properties provide good prosthesis retention. Excellent resistance to oral environment and mechanical strength assure good long-term performance. Low film thickness of cement. Versatile in use:

- Temporary cement on natural abutment teeth.
- Semi-definitive cement for crowns on implants.
- Temporary cement for definitive aesthetic elements (veneers, onlays), due to its translucent beige shade.

## Safety notes

### Warnings

- Contains polymerizable methacrylate resins which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.

### Precautions

- The use of this product is restricted to dental professionals.
- The product must not be used for any purpose other than specified in the indications.
- Wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients.

## Contraindication

Product is contraindicated in patients with a history of allergic reaction to methacrylate resins.

## Composition

Hydrophilic dimethacrylates - modified urethane dimethacrylates - amorphous silica - Camphorquinone (CQ) - accelerators - stabilizers - pigments.

## Clinical procedures

### First use of a new automix-syringe

- Remove and discard the cap of the automix-syringe. Under no circumstances should the cap of the automix-syringe be used to reseal the automix-syringe as this could result in the inclusion of air bubbles, inducing a cross-contamination [base/catalyst] and leading to a premature polymerization of the product.
- Check whether both automix-syringe openings are free and remove obstructing material, if any, with a suitable instrument.
- Before installing a mixing tip, bleed the automix-syringe onto a mixing pad until the base and catalyst pastes are at the same level.

### Seal Temp S, automix-syringe form

1. Remove automix-syringe cap, dispense and discard a small amount of material. Be sure the material is flowing freely from both ports.
2. Install a mixing tip on the automix-syringe
3. Gently depress syringe plungers, to begin the flow of material. Do not use excessive force. Extrude enough material to make a thin layer of cement inside the temporary crown.
4. Hold the temporary crown firmly in place.

5. Trim excesses immediately.
6. Keep the automix-syringe with the mixing tip in place until the next use.

### Seal Temp S, 2 syringes form

1. Dispense approximately equal amount of Part A and Part B pastes onto a mixing pad.
2. Mix the pastes thoroughly with a spatula for 10 to 15 seconds
3. Spread a thin layer of cement inside the temporary crown and hold the crown firmly in place.
4. Trim excesses immediately.

## Notice

Seal-Temp S is retentive enough for normal height abutment teeth.

## Storage & shelf-life

- The product must be used by the expiration date indicated on the package.
- Do not expose the product to sun-light or warm sources.
- Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it should be brought back to room temperature 15 minutes minimum before use, in order to restore its normal viscosity and curing time.
- Use the product at room temperature.

## Important notice

- Leave the used mixing tip in place on the automix-syringe until next use.
- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.

## Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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