HARMONY GB

Definition

Harmony vinylpolysiloxane impression materials have been designed to make high precision dental impressions. The extreme hydro-compatibility of the material, its mat appearance and its thixotropy, makes it one of the most performing impression materials available, in term of clinical features.

Safety notes

Warnings

- Contains polymerizable monomers and surfactants which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs immediately remove material with cotton and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid ingestion/swallowing of material to prevent irritation or obstruction. If accidental swallowing occurs, drink lots of water. This material is not hazardous when small quantities are ingested. Larger quantities may cause bowel obstruction. Seek medical attention in the event of digestive irregularities.

Precautions

- The use of this product is restricted to dental professionals.
- The product must not be used for any purpose other than specified in the indications.
- Wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients.
- Higher temperatures reduce working times and setting times (faster), lower temperatures increase them (slower).
- To obtain optimum physical properties, the mix should be completely homogeneous.
- Should be used with properly fitted, rigid trays for most applications. Trays with retentive features (perforations), are recommended.
- Material should extrude easily : do not use excessive force. Excessive pressure may result in unanticipated extrusion of the material or cause cartridge rupture.

Contraindication

Product is contraindicated in patients with a history of allergic reaction to vinylpolysiloxane materials.

Composition

Vinylpolysiloxane - Highly dispersed silica - Cosmetic pigment -Paraffine oil - Platinum complex and metal - Titanium dioxide.

Clinical procedures

First use of a new cartridge

- Remove and discard the cap of the cartridge. Under no circumstances should the cap of the cartridge be used to reseal the cartridge as this could result in the inclusion of air bubbles, inducing a cross-contamination (base/catalyst) and leading to a premature polymerization of the product.
- Check whether both cartridge openings are free and remove obstructing material, if any, with a suitable instrument.
- Before installing a mixing tip, bleed the cartridge onto a mixing pad until the base and catalyst pastes are at the same level

Impression

- Remove and discard the used mixing tips.
- Attach a new mixing tip (yellow ring type) and an intraoral tip (yellow), on the cartridge.
- Place the cartridge into the dispensing gun.

One-step technique

Recommended materials: HPN-600 and HM-100/HM-200/HM-40

- Gently remove the retraction cord, or wash off retraction material, from the sulcus of the prepared teeth. Flush and dry.

- Prepare the putty material by manually mixing the base and catalyst pastes (50/50) and load a perforated metallic impression tray.
- While the tray is being loaded, apply the wash material around the cleaned and dried preparations[1].
- Depending on the number of teeth involved, initiate application during the putty preparation, so the tray loading and application of material around the preparation are completed at the same time. Be careful not to exceed the working time of the material to avoid incomplete or inaccurate impressions.
- Slowly place the loaded tray in the mouth and hold it in position passively.
- Allow the impression material to fully set, then remove the impression from the mouth.
- Wash the impression thoroughly and place it in a disinfectant solution, prior to send it to the technician laboratory. The duration should comply with the manufacturer IFU of the solution.

Two-steps technique ["wash technique"]

Recommended materials: HPF-600 and HLN-100/HLN-200/HLN-40 or HLF-100/HLF-200/HLF-40

- Prepare the putty material by manually mixing the base and catalyst pastes (50/50) and load a perforated metallic impression tray.
- Gently remove the retraction cord, or wash off retraction material, from the sulcus of the prepared teeth. Flush and dry.
- Slowly place the loaded tray in the mouth and hold it in position.
- Allow the impression material to fully set, then remove the impression from the mouth.
- Excise undercut and silicone present in the inter-dental space, and cut spillways, if required.
- Clean and dry the impression thoroughly to provide for optimal adhesion between the putty and the light materials.
- Insert the light impression material into the preliminary impression at the appropriate sites and apply around the cleaned and dried preparations, while keeping the intra-oral tip immersed in the paste, to avoid the formation of bubbles.
- Replace the preliminary impression into the mouth and hold it in position passively.
- Allow the impression material to fully set, then remove the impression from the mouth.
- Wash the impression thoroughly and place it in a disinfectant solution, prior to send it to the technician laboratory. The duration should comply with the manufacturer IFU of the solution.

Storage & shelf-life

- The product must be used by the expiration date indicated on the package.
- Do not expose the product to sun-light or warm sources.
- Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it should be brought back to room temperature 15 minutes minimum before use, in order to restore its normal viscosity and curing time.
- Use the product at room temperature.

Important notice

- Save the used mixing tip in place on the cartridge until next use.
- Do not touch the impression material with latex gloves. They can inhibit its polymerization.
- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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