## PF Seal

**GB**

### Definition

Pits and fissures sealant

### Safety notes

#### Warnings

- PF Seal contains polymerizable methacrylates which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.

#### Precautions

- Wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients.

### Contraindications

PF Seal is contraindicated in patients with a history of allergic reaction to methacrylate resins or any other of the components. Do not use PF Seal in cases of pulp exposure, pulpitis.

### Instruction for use

- Prepare the surface of the tooth by prophylaxis, using a non-fluoride prophylactic-paste. Wash and dry.
- Apply etching gel onto the enamel. Rinse for 20 seconds with water. Dry it in with a water and oil-free air. Properly etched tooth should have a dull, chalky-white appearance. Highly mineralized tooth may require an additional one to two minutes etching to achieve this effect.
- Isolate the tooth to be restored from salivary contamination (rubber dam).
- Apply a little quantity of PF SEAL at the dry surface of the fissure. Using needle tip or a disposable brush (Fine or XFine size), push the sealant down to the bottom of the fissure.
- Light cure with a lamp (min. power: 500 mW/cm<sup>2</sup>) for 20 seconds. Hold the nose of the lamp as close as possible to the surface of the composite.
- Finish and polish the restoration material.

### Storage & shelf-life

Do not expose the product to sun-light or warm sources. Store at a temperature not exceeding 25°C. When stored under such conditions, PF Seal has a shelf-life of 36 months.

### Presentations & reference

White, opaque colour.  
PFS-3,6 - 2 x 1,2ml syringes + 10 needle tips

### Important notice

- Do not throw away the box and the instruction for use until the product is completely finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

### Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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