# Elsotemp

**GB**

## Definition

Zinc oxide and synthetic materials temporary filling cement, water setting, ready to use. No expansion when setting, due to the high-quality synthetic materials used. Eugenol-free to not hinder polymerization of resins and composites. Contains fluoride for a cariostatic action.

## Safety notes

### Warnings

- Contains Zinc Sulfate which may seriously irritate eyes.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.
- Harmful if swallowed. Don't give anything to drink. Do not induce vomiting and seek for medical advice.

### Precautions

- The use of this product is restricted to dental professionals.
- The product must not be used for any purpose other than specified in the indications.
- Wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients.

### Contraindication

Product is contraindicated in patients with a history of allergic reaction to Zinc Sulfate.

### Composition

Cataplastic (resin) – Plaster – Zinc sulfate – Zinc oxide

### Clinical procedures

1. Remove the desired amount of cement with a spatula. Tightly close the jar after use.
2. Apply the cement in the cavity, with a spatula, hardening start immediately with saliva.
3. When needed, remove it with an ultrasonic device or with a probe.

### Storage & shelf-life

- The product must be used by the expiration date indicated on the package.
- Do not expose the product to sun-light or warm sources.
- Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it should be brought back to room temperature 15 minutes minimum before use, in order to restore its normal viscosity and curing time.
- Use the product at room temperature.

### Presentations & references

Three consistencies available: grey[soft] - white[normal] - pink[hard]

### Important notice

- Do not throw away the box and the instruction for use until the product is finished.

- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

### Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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