Description

NOTREDAM is a methacrylate-based light-cure barrier (liquid dam), used in dental procedures. It is hard when set, however easy to remove from the gum, dentin walls, undercuts and embrasures

Bleaching

To protect gum during bleaching sessions at the chair.

Endodontic

To fill and hermetically seal the gap between the dam and the tooth during endodontic sessions.

To built a temporary missing wall of the tooth, to keep the irrigation liquid in the operative field.

Prosthesis

To block undercuts, in case of receding gum, before impression.

Safety notes**Warnings**

- Contains polymerizable methacrylate resins which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.
- Avoid ingestion of the product. If signs of digestive disorder appear, contact a doctor.

Precautions

- The use of this product is solely restricted to dental personnel.
- The product must not be used for any purpose other than specified in the indications.
- For dental professionals wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients.
- Isolation of the operating field is requested (e.g. dental dam).
- Dispose according to the national and/or federal standards in force.

Contraindication

Product is contraindicated in patients with a history of allergic reaction to any of the components.

Importante notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Incompatibilities

The product should not be used with eugenol-containing materials for pulp protection or temporary sealing since the eugenol can retard the curing process.

Composition

Dimethacrylate resins (80-95%) - Glass fillers (1-5%) - Camphorquinone (<1%) - Accelerators (<1%) - BHT (<1%) - Pigments (<1%)

Clinical procedures**Bleaching**

1. Remove the cap from the luer lock syringe and secure a disposable applicator.
2. Rinse thoroughly the designed area and slightly dry the surface of the teeth, gingival third, the embrasures or the gaps to fill.
3. Apply NOTREDAM onto the gingival and the embrasures. If some product flows in excess onto the enamel, cure the excess for 5 seconds and use a mouth spatula to remove it. For the embrasures, be sure that the resin injected from the vestibular side will appear on the lingual side. This extension will add retention to the resin barrier.
4. When the barrier is applied for bleaching, extend the resin barrier beyond the last tooth to be bleached.
5. The resin barrier should be about 5-6mm height and

2. mm thick.
- Light-cure for 20 seconds.

Clean-up procedure

1. Rinse thoroughly the treated area to eliminate the whitening gel and vacuum.
2. Lift the polymerized dam, using a probe or a spatula. The interproximal dam is eliminated by using a probe and/or floss.
3. Rinse again the gum and interproximal areas.

Note

This device is used on young adults to elderly people.

Storage & shelf-life

- The product must be used by the expiration date indicated on the package.
- Do not expose the product to sun-light or warm atmosphere.
- Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it should be brought back to room temperature 15 minutes minimum before use, in order to restore its normal viscosity and curing time.
- Use the product at room temperature.

Important note

- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

Disclaimer

Disposal must be carried out in accordance with local, regional and national regulations in force. These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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