# PURESHADE

GB

## Description of the product

Light-cure nano-hybrid composite, universal, for anterior and posterior restorations of class I, II, III, IV and V. BIO+ formulation BPA-free (no BisGMA/BisEMA), TEGDMA-free and no addition of HEMA.

### Safety notes

#### Warnings

- Contains polymerizable methacrylate resins which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.
- Avoid ingestion of the product. If signs of digestive disorder appear, contact a doctor.

#### Precautions

- The use of this product is restricted to dental personnel.
- The product must not be used for any purpose other than specified in the indications.
- Wear suitable protective eyewear, mask, clothing and gloves for dental professionals.
- Protective eyewear is recommended for patients.
- Isolation of the operating field is requested (e.g. dental dam).
- Dispose according to the national and/or federal standards in force.

#### Contraindication

Product is contraindicated in patients with a history of allergic reaction to any of the components.

#### Important notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

#### Incompatibilities

The product should not be used with eugenol-containing materials for pulp protection or temporary sealing since the eugenol can retard the curing process.

#### Composition

Dimethacrylate resins (15-25%) - Barium alumino silicate, specific fillers (70 -80 %) - Camphorquinone (<1%) - Accelerators (<1%) - BHT (<1%) - Pigments (<1%)

## Shade selection

| ELSDENT SHADES* | VITA SHADES                   |
|-----------------|-------------------------------|
| Shade 1         | A1 - B1 - B2 - C1 - D2        |
| Shade 2         | A2 - A3 - C2 - D3 - D4        |
| Shade 3         | A3.5 - A4 - B3 - B4 - C3 - C4 |

\*Shade 1 = light shades - Shade 2 = medium shades - Shades 3 = dark shades

#### Clinical procedures

1. Select the appropriate shade with daylight.
2. Prepare the tooth cavity in a conventional manner. The margins are prepared by slightly beveling the occlusal surfaces and producing slight flares on interproximal cavities.
3. Isolate the operating field from salivary contamination (rubber dam).
4. For classes II, III and IV, a matrix band is placed and secured by wedges; it should be loosened to allow full contours when packing the restorative material into the cavity.
5. If the cavity is closed to the pulp, apply a eugenol-free cavity liner (ex: calcium hydroxide, antibacterial liner).
6. Etch the enamel and dentin.
7. Apply the light curing bonding agent of your choice, following manufacturer's instructions.
8. Apply Pureshade
9. Polymerize Pureshade in 2mm layer, using a Halogen light (700/800mW/cm2) or a LED light (>1200 mW/cm2).
10. Exposure time is 20 seconds for light shades (A1, B1, B2, C1, C2), medium (A2, A3, C2, D3, B3) and dark shades (A3, A3.5, A4, B4, C3, C4, D4).
11. Finish and polish the restoration.

#### Note

This device is used on young adults to elderly people

#### Storage & shelf-life

- The product must be used by the expiration date indicated on the package.
- Do not expose the product to sun-light or warm sources.
- Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it should be brought back to room temperature 15 minutes minimum before use, in order to restore its normal viscosity and curing time.
- Use the product at room temperature.

#### Important notice

- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

## Disclaimer

Disposal must be carried out in accordance with local, regional and national regulations in force. These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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