ULTIMA

MODE D'EMPLOI

USER'S GUIDE

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PRODUCT INFORMATION

The ULTIMA curing light is wireless pen-style, polymerization lamp for use by trained dental professionals in dental offices or dental laboratories.

PRECAUTIONS

Do not look directly into the light output. Patient, clinician and assistants should wear UV orange eye protection when this device is in use. Light reflected from the tooth surface may also injure eyes. Use the ULTIMA Shields supplied with the unit.

- Do Not expose soft oral tissues at close proximity. Maintain a safe distance between the lens and the soft tissue.
- Never modify ULTIMA or any of its equipment on your own. Any modification may compromise safety and effectiveness. Should any problem occur, seek qualified help from your distributor.
- This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.
- Any Dentist with a history of retinopathy should consult their eye specialist before operating this unit. Use the ULTIMA curing light extremely carefully and comply with all the necessary safety precautions (including wearing suitable, light filtering safety glasses).
- Anyone who has had a cataract operation may be especially sensitive to light and should be advised against undergoing treatment with an ULTIMA curing light unless adequate safety precautions are taken such as wearing suitable, light filtering safety glasses.
- Do not use when patients are prone to photobiological reactions (including patients with solar urticaria or erythropoietic protoporphyria) or those currently undergoing treatment with photosensitizing pharmaceuticals.
- Do not use a ULTIMA curing light which has not been properly cleaned and disinfected.
- Protect ULTIMA curing light from cross-contamination by applying Disposable Barrier Sleeves. Sleeves are intended for single use only. Discard after use. Do not reuse sleeves in other patients in order to prevent cross-contamination.
- For electrical medical devices specific safety precautions apply which have to be observed. Do not use mobile phones or other portable communication tools near the curing light while in use, as they could interfere with the function of the ULTIMA curing light.

- All dental curing lights cause a certain degree of heat development. Extended operation in areas near the pulp and soft tissues may result in severe damage. Do not cure the same tooth surface for more than 20 seconds at a time. If required, polymerize at intermittent intervals of 20 seconds each.

ULTIMA FEATURES

The ULTIMA is characterized by :

- Small size and lightweight ergonomic design.
- Blue and UV light emission combined to a single beam suited for any type of dental composite.
- The curing light with a white light mode for diagnostics.
- Compact cordless design with convenient handling features and exchangeable off-the-shelf battery.
- Single piece rugged aluminum body, specifically coated for dental use.
- Thin tip design to reach the most difficult areas inside the oral cavity.
- Several power modes to cover different treatment procedures.
- Advanced heat management system limiting LED tip temperature.
- Indication panel with two buttons for simple operation.

OPERATION AT A GLANCE

- Open box and take out the parts. Make sure they all parts are intact.
- Open Battery compartment by rotating the cap counterclockwise and pulling outwards.
- Insert battery to the battery compartment, "+" side facing outwards. Make sure that after the battery is inserted you can see the "+" side of the battery.
- Close the battery cap by pushing it into its slots and rotating clockwise. Make sure the cap is aligned with the handpiece. Handpiece will sound a signal and the indication LEDs will turn on for 1 second.
- You can immediately operate your ULTIMA on normal mode by pushing the ON/OFF button.
- To change modes before operating, press the mode button to cycle between the different modes.

ELEMENTS OF THE KIT

ULTIMA curing light

Designation	Quantity
ULTIMA handpiece curing light	1
ULTIMA charging station: Base, USB to Micro-B cable, power adaptor	1
Orange eye protection cap	2
Disposable protection sleeves	100
Standard Li-Ion rechargeable battery	1



Elements of the kit and optional elements - Ref numbers for reorder

Designation	Reference	Quantity
ULTIMA handpiece curing light - Aluminium shade	ULTI-G	1
ULTIMA handpiece curing light - Black matt shade	ULTI-N	1
Standard single battery USB charger	ULTI-C-BAT	1
Orange eye protection cap	ULTI-PO	2
Disposable protection sleeves	ULTI-G500	500
Standard li-ion rechargeable battery	ULTI-BAT-3.7L	1

STEP-BY-STEP INSTRUCTIONS


Charging my ULTIMA with the Charging Station

1. Use 5V 2.0A charger that is included in kit.
2. Attach 5V adapter to electric outlet, and connect USB cable to power jack.
3. Insert the handpiece into the charging Station. There will be three rapid beeps and the battery light will be flashing green, indicating battery is recharging.
4. When fully charged your handpiece will show a constant green light.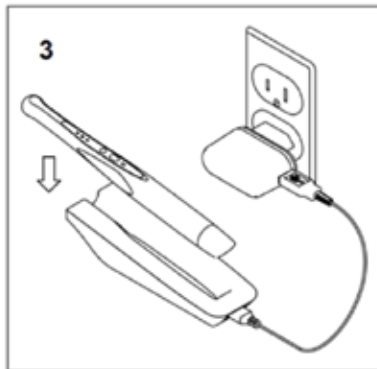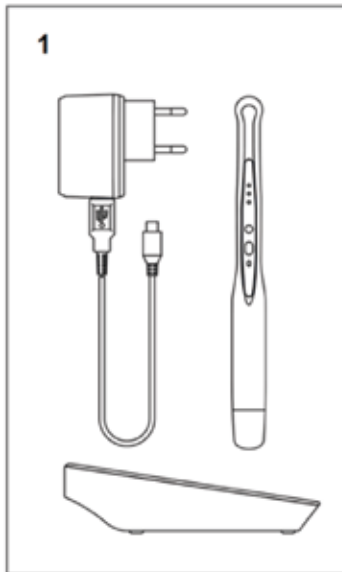

OPERATING DIFFERENT MODES


	▶ SOFT	▶ STANDARD	▶ BOOST	▶ SEQUENTIAL BOOST	▶ UV DIAGNOSIS	▶ TRANSILLUMINATION
W	⊗	⊗	●	⊙	⊗	●
UV	⊙	●	●	⊙	●	⊗
B	⊙	●	●	⊙	⊗	⊗
	10s	20s	3s	10x (3s ON + 2s OFF)	120s	120s

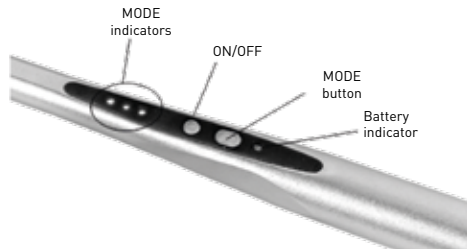- ⊗ OFF
- ON fix
- ⊙ ON blinking
- ▶ Press MODE button

How to change batteries ?

1. Turn cap counterclockwise until loose and extract battery.
2. Insert new battery "minus" side first. Insert cap and turn clockwise until locked.

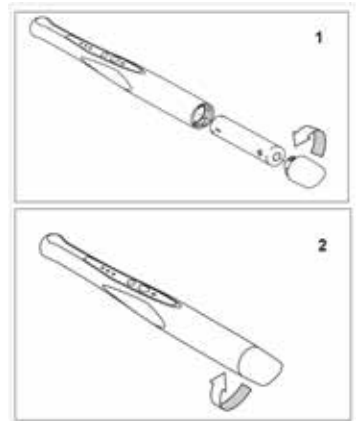Storage for long period of time :

Remove battery, close battery cap, store all parts in a dry, dark, and cool place.

MAINTENANCE

The ULTIMA curing light is a sealed unit with a sapphire-like hard surface and a scratch resistant glass lens.

- After each use, moisten a gauze or soft cloth with an anti-microbial surface disinfectant and wipe the surface and lens. (see next section acceptable cleaners)
- Periodically check the lens for cured dental resins. If necessary, use a non-diamond dental instrument to carefully remove any adhered resin.
- Barrier sleeves prevent cross contamination and helps keep dental composite material from adhering to the surface of the lens and wand body.

NOTE : Lightmeters differ greatly and are designed for specific light guide tips and lenses. The true numeric output will be skewed due the inaccuracy of common light meters and therefore it is recommended to compare the power of the device to itself through time in order to maintain LED power output. We recommend regular checking of your ULTIMA in Standard power mode.

WARNING - CAUTION

- Do not autoclave batteries, charging station, power adapter, or ULTIMA Handpiece.
- Do not insert fingers, instruments, or other objects into the battery compartment of the ULTIMA.
- Do not attempt to clean the nickel plated contacts, or any part of the battery compartment. Call customer service if you have any concerns.
- Do not immerse in any kind of ultrasonic bath or any liquids.
- Do not wipe down the ULTIMA curing light with caustic or abrasive cleaners.
- Do not wipe down the ULTIMA curing light with wipes too impregnated or too wet.
- Do not use directly spray-applied products.

See lists of acceptable cleaners below.

ACCEPTABLE CLEANERS

- Cavicide™ products
- Isopropyl alcohol-based cleaners
- Ethyl alcohol-based cleaners
- Lysol® disinfectant
- Other non-bleach and non-abrasive disinfectants or cleaners

DO NOT USE

- Wipes too impregnated or too wet.
- Strong alkali detergent of any type, including hand soaps and dish soaps.
- Bleach-based cleaners (e.g. - Clorox™, Sterilox™)
- Hydrogen Peroxide based cleaners
- Abrasive Cleansers (e.g. - Comet Cleanser™)
- Acetone-based cleaners (e.g. - nail polish remover, Goo-off™)
- MEK (Methyl Ethyl Ketone)

BATTERY REPLACEMENT

Authorized batteries for use which have been checked by the manufacturer are:

- AWT 14650 IMR 1100mAh
- Trustfire 14650 IMR 900mAh

HYGIENE

ULTIMA Barrier Sleeves

- The use of the ULTIMA Sleeves is a mandatory precautionary measure against cross contamination, but does not substitute reprocessing of the ULTIMA curing light.

Cross-contamination - Barrier sleeves are intended for single use only

Discard Barrier Sleeves according to local regulations

ULTIMA Handpiece

Right after use carefully remove sleeve without contaminating device and discard sleeve according to local regulations. Thoroughly wipe clean device with appropriate disinfectants mentioned in the Maintenance section.

Cross-contamination - always make sure handpiece is completely reprocessed before inserting to charging station

ULTIMA Eye protection Cap

Eye protection cap can be autoclaved.


Environmental Note

Device and battery disposal

This device is provided with a Li-ion battery. Device and battery must not be disposed in normal domestic waste. For environmental reasons, please dispose device and battery according to local environmental guidelines or regulations.

TECHNICAL DATA

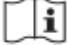AC supply connection	100 V...240 V / ~ 50 Hz...60 Hz
Power input	Maximum 14 W
Equipment class	Charging station Class II (IEC 60601-1)
Follow instructions for use	
Protection from ingress of liquids	Handpiece : IPX4 Charging station : IP20
Operation	Ambient temperature : Between 0 °C and +30 °C (32 °F to 86 °F)
Relative humidity	Between 20% and 90%
Storage	Ambient temperature : Between -10 °C and +35 °C (14 °F to 95 °F)
Transport :	Ambient temperature : Between -10 °C and +50 °C (14 °F to 122 °F)
Battery Protection	While charging - thermistor
Light intensity	Soft Standard Boost Sequential Boost Diagnosis UV White Light
Wavelength	385 to 480 nm
Handpiece weight	95 grs (inc. battery)
Handpiece dimensions	Length 210mm. max diameter 20mm.
Charging station dimensions	Length 152mm x width 50mm x height 42mm
Warranty	4 years
Medical Device	MD Class I

Compliance

- 2017/745 Medical Device Regulation (EU)
- 2002/95/EC Waste Electrical and Electronic Equipment (WEEE II)
- IEC 60601-1 – Medical Electrical Equipment (General requirements for basic safety and essential performance)
- IEC 60601-1-2 – Medical electrical equipment – Part 1-2: General requirements for safety and essential performance – Collateral standard: Electromagnetic compatibility – Requirements and tests
- ISO 10650 – Dentistry – Powered polymerization activators
- EN 62471 – Photobiological Safety of Lamps and Lamp Systems

WARRANTY TERMS

Serial number

Handpiece and Charging station have different serial numbers. Handpiece serial number can be found inside the handpiece viewed from the battery entrance. Charging station serial number is on the label on its bottom side. **The serial number should be quoted in all correspondence, which requires identification of the product.**

Warranty terms.

We hereby grant a 4-years warranty on all components of the ULTIMA except the battery. The battery is covered by a 6 months warranty. The warranty commences on the date of purchase.

Within the warranty period, we will eliminate free of charge any defects in the appliance resulting from faults in material or workmanship either by repairing or exchanging parts or exchanging the whole device at our discretion.

Not covered by this warranty:

Damage arising from improper use (operation with incorrect current or voltage, unsuitable power outlet, breakage, cleaning by other than the recommended methods), normal wear and defects which have a negligible effect on the value, function or operation of the appliance. This warranty becomes void if repairs are undertaken by unauthorized persons.

Service under this warranty does not affect the expiration date of the warranty. The warranty on parts or entire devices which are exchanged ends when the warranty on this device expires.

In the event of a claim of this device, return the complete device (charging station and the LED curing light) together with the invoice to your dealer or distributor.

All other claims including those for damages resulting are excluded unless our liability is legally mandatory.

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