CORE D & CORE D Flow GB

Description of the product

Dual-cure composites for core build-up of devitalized teeth.

Physical data/Characteristics of the material

- Working time : 2'20" - 2'40" [22°C]
- Intra oral setting time + working time (self-curing model): 3'10" - 4'10" [22°C]
- Flexural strength : 135 Mpa
- Compressive strength : 330 Mpa

Safety notes

Warnings

- The products contain methacrylate resins which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.
- Avoid ingestion of these products. If signs of digestive disorder appear, contact a doctor.

Precautions

- Use of the products are restricted to dental personnel.
- Products must not be used for any purpose other than specified in the indications.
- Wear suitable protective eyewear, mask, clothing and gloves for dental professionals.
- Protective eyewear is recommended for patients. Isolation of the operating field is requested (e.g. dental dam).
- Dispose of these products according to the national and/or federal standards in force.

Contraindication

Products are contraindicated in patients with a history of allergic reaction to any of the components.

Important notice

Any serious incident that has occurred in relation to the products should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Incompatibilities

Products should not be used with eugenol-containing materials for pulp protection or temporary sealing since the eugenol can retard the curing process.

Composition

Dimethacrylate resins (25-35%) - Glass fillers (50-70%) - Camphorquinone (<1%) - Accelerators (<1%) - BHT (<0,1%) - Pigments (<1%)

Clinical procedures

First use of a new cartridge/syringe

- Remove and discard the cap of the cartridge/syringe. Under no circumstances should the cap of the cartridge be used to reseal the cartridge/syringe as this could result in the inclusion of air bubbles, inducing a cross-contamination (base/catalyst) and leading to a premature polymerization of the product.
- Check whether both cartridge/syringe openings are free and remove obstructing material, if any, with a suitable instrument.
- Before installing a mixing tip, bleed the cartridge/ syringe onto a mixing pad until the base and catalyst pastes are at the same level.

- Store at a temperature between 5°C and 25°C. When the products are stored in the refrigerator, they should be brought back to room temperature 15 minutes minimum before use, to restore their normal viscosity and curing time.
- Use the products at room temperature.

Important notice

- Leave the used mixing tip in place on the cartridge/ syringe until next use.
- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

Disclaimer

Dispose of the product in accordance with applicable local, regional and national regulations. This material is intended exclusively for use in dentistry. Processing must be done strictly in accordance with the instructions for use. No liability can be accepted for damage caused by failure to comply with the instructions for use or the prescribed area of application. It is the responsibility of the user to check the material for its suitability and use for a purpose not expressly stated in the instructions for use. Descriptions and information do not constitute a guarantee of properties and are not binding.

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