ELSOTEMP

GB

Description of the product

Zinc oxide based temporary filling cement with fluoride, self-hardening, ready to use. Low expansion when setting, due to the high-quality synthetic materials used. Eugenol-free to not hinder polymerization of resins and composites. Excellent resistance to masticatory pressure.

Indication

Temporary filling cement for standard cavities and after endodontic treatments.

Safety notes

Warnings

- Contains Zinc Sulfate which may seriously irritate eyes.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention. Avoid prolonged inhalation
- Avoid prolonged or repeated contact with skin to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.
- Avoid ingestion of the product. If signs of digestive disorder appear, contact a doctor.

Precautions

- The use of this product is restricted to dental personnel.
- The product must not be used for any purpose other than specified in the indications.
- For dental professionals, wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients.
- Dispose according to the national and/or federal standards in force.

Contraindication

Product is contraindicated in patients with a history of allergic reaction to any of the components.

Importante notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Incompatibilities

Interactions with other agents are unknown.

Composition

Plasticizer [5-15%] - Plaster [20-30%] - Zinc sulfate [5-20%] - Zinc oxide [5-20%]

Clinical procedures

- Take the required quantity of Elсотemр with a suitable instrument [mouth spatula]. Tightly close the jar after use.
- Apply the cement in the cavity. Hardening start immediately in contact with saliva.
- Remove excess by wiping it off with a moist cotton pellet.
- Elсотemр will harden totally in about 40 minutes [water-setting].
- Ask the patient to avoid chewing for at least 1 hour.
- To remove the temporay cement, use an ultrasonic scaler or a probe.

ELSOTEMP

FR

Note

This device is used on young adults to elderly people.

Storage & shelf-life

- The product must be used by the expiration date indicated on the package.
- Close the jar tightly after use.
- Do not expose the product to sun-light, warm sources or humidity.
- Store at a temperature between 5°C and 25°C.
- Use the product at room temperature.

Presentations & references

Three consistencies available: grey[soft] - white[normal] - pink[hard]

Important notice

- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.
- Do not mix or contaminate with any other materials

Disclaimer

Disposal must be carried out in accordance with local, regional and national regulations in force. These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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