G-ETCH

GB

Description of the product

G ETCH is a dental etching gel at 38% phosphoric acid. Non-drying, medium viscosity gel, thixotropic, washes off quickly and easily, excellent water solubility, very good contrast.

Safety notes

Warnings

- Contains phosphoric acid which may cause respiratory irritation, severe skin burns and eye damage.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention. If accidental burn were occurred, immediate wash with copious water. And bring the patient to the dermatologist as soon as possible.
- Harmful if swallowed. Rinse mouth thoroughly. Do not induce vomiting and seek for medical advice or call a Poison Center.

Precautions

- The use of this product is solely restricted to dental personnel.
- The product must not be used for any purpose other than specified in the indications.
- Dental professional should wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients.
- Dental acid-etchants can cause irreversible injury to the cells when they come in direct contact with them, even at low concentrations and for short application times. To maximize the function of dental acid-etchants and minimize the adverse effects on soft tissue, it is important to prevent the inflow of dental acid-etchants into soft tissues by exposing only the treatment area. The dentist should put in place a dental rubber dam to thoroughly isolate the operating field from soft tissue, and use of gel-type dental acid-etchants. For dental hygienists, it is important to use powerful suction system with a large size suction tip to eliminate completely acid-etchants so that residual acid-etchants do not irritate the oral mucosa.
- Dispose according to the national and/or federal standards in force.

Contraindication

G-Etch is contraindicated in patients with a history of allergic reaction to/or hypersensitivity to acids or to any of the components.

Important notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Incompatibility

G-Etch should not be mix with sodium hypochlorite, mixing will generate Chloride gas which is harmful.

Composition

Water [35-50%] - Phosphoric acid (38%) - Glass filler (2-5%) - Pigments (<1%)

Clinical procedures

- Prepare tooth by cleansing the enamel surface to be etched with a slurry of pumice without oil or fluoride.
- A dental dam should put in place, or any other solution to isolate the operating field, is requested.
- Apply G-Etch on both enamel and dentin. Recommended time is 20 seconds maximum on dentin and 40 seconds on enamel.
- Rinse off with copious amount of water, and vacuum at the same time. Do not contaminate with other fluids like saliva.

- Leave surface wet, or dry surface with uncontaminated air, depending on the instructions for use for the bonding you are using. Etched enamel should appear chalky white. If not, reapply G-Etch following points 2 and 3 again.
- Cap syringe immediately after use.

Filling method for the jumbo kit

Important notice

Wearing protective goggles is compulsory during the filling operation.

- Remove caps protecting syringe ends.
- Prepare the 3ml syringe for loading by depressing plunger to the end of the barrel.
- Secure the ends of the two syringes (the empty 3ml one and the 50ml bulk pack storage syringe) together by interlocking them, using the luer-lock coupling device; hold barrel of the 50ml syringe, securely hand screw the empty 3ml syringe with the other hand by turning clockwise. Be sure the fit is tight but do not force.
- Pull the plunger of the dispensing syringe (3ml) to fill it. Never push on the plunger of the storage syringe (50ml).
- When the dispensing syringe (3ml) is full, slightly push its plunger to avoid any leaking when separating the 2 syringes from each other.
- To disengage syringes, grip the luer-lock with one hand and with the other one, turn the barrel of the 3ml syringe anti clockwise.
- Attach the appropriate protective caps on both syringes.
- To apply etching gel, remove cap from the 3ml syringe and insert a disposable needle application tip.

Storage & shelf-life

- The product must be used by the expiration date indicated on the package.
- Do not expose the product to sun-light or warm sources.
- Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it should be brought back to room temperature 15 minutes minimum before use, in order to restore its normal viscosity and curing time.
- Use the product at room temperature.

Important notice

- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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