Description of the product

Light-cure resin-based liner.

Safety notes**Warnings**

- Contains polymerizable methacrylate resins which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.
- Avoid ingestion of the product. If signs of digestive disorder appear, contact a doctor.

Precautions

- The use of this product is restricted to dental professionals.
- The product must not be used for any purpose other than specified in the indications.
- Wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients.
- Isolation of the operating field is requested (e.g. dental dam).
- Dispose according to the national and/or federal standards in force.

Contraindication

Product is contraindicated in patients with a history of allergic reaction to any of the components.

Important notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Incompatibilities

The use of eugenol-containing products with dental materials based on composites should be avoided, since they interfere with the curing process.

Composition

Dimethacrylate resins (40-60%) - Glass fillers (30-50%) - Camphorquinone (<1%) - Accelerators (<1%) - BHT (<0,1%) - Pigments (<1%)

Polymerization Time

HISEAL is a light-curing composite-based product, and therefore also sensitive to natural or artificial light. The working time is significantly shorter when used under artificial light.

Clinical procedures

- Isolate the operating field (e.g. dental dam).
- Prepare, rinse and dry the cavity.
- Isolate treatment area from salivary contamination [cotton rolls, rubber dam etc...]

- Apply HISEAL directly over the dentin using a disposable applicator. The first coating should not be more than 0.5 mm thick. Remove any material inadvertently applied over enamel margins. Keeping the tip as close to the cured surfaces as possible, polymerize, using a Halogen/LED light (500/1000mW/cm²) or a LED light (>1000 mW/cm²)

- If indicated, apply a second layer of HISEAL, with maximum thickness of 1.5mm. Light-cure for 20 seconds.
- Restore the tooth with a composite, per manufacturer instructions. HISEAL is compatible with all commercially available dental restorative materials.

Note

If using HISEAL under water-based cements such as glass ionomer, remove any uncured resin from the HISEAL surface by wiping it with a pledge of cotton moistened with alcohol.

The product is used on young adults to elderly people.

Storage & shelf-life

- The product must be used by the expiration date indicated on the package.
- Do not expose the product to sun-light or warm sources.
- Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it should be brought back to room temperature 15 minutes minimum before use, in order to restore its normal viscosity and curing time.
- Use the product at room temperature.

Important notice

- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

Disclaimer

Dispose of the product in accordance with applicable local, regional and national regulations. This material is intended exclusively for use in dentistry. Processing must be done strictly in accordance with the instructions for use. No liability can be accepted for damage caused by failure to comply with the instructions for use or the prescribed area of application. It is the responsibility of the user to check the material for its suitability and use for a purpose not expressly stated in the instructions for use. Descriptions and information do not constitute a guarantee of properties and are not binding.

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