PURESHADEGB

Description of the product
Light-cure nano-hybrid composite, universal, for direct restorations. Complies with ISO 4049 for polymer-based dental restorative materials. BIO+ formulation BPA-free (no BisGMA/BisEMA), TEGDMA-free and no addition of HEMA.

- Safety notes**
Warnings
- Contains polymerizable methacrylate resins which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
 - Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
 - Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
 - Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.
 - Avoid ingestion of the product. If signs of digestive disorder appear, contact a doctor.

- Precautions**
- The use of this product is restricted to dental personnel.
 - The product must not be used for any purpose other than specified in the indications.
 - Wear suitable protective eyewear, mask, clothing and gloves for dental professionals.
 - Protective eyewear is recommended for patients.
 - Isolation of the operating field is requested (e.g. dental dam).
 - Dispose according to the national and/or federal standards in force.

Contraindication
Product is contraindicated in patients with a history of allergic reaction to any of the components.

Important notice
Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Incompatibilities
The product should not be used with eugenol-containing materials for pulp protection or temporary sealing since the eugenol can retard the curing process.

Composition
Dimethacrylate resins (15-25%) - Barium alumino silicate, specific fillers (70 -80%) - Camphorquinone (<1%) - Accelerators (<1%) - BHT (<0.1%) - Pigments (<1%)

ELSODENT SHADES*	VITA SHADES
Shade 1	A1 - B1 - B2 - C1 - D2
Shade 2	A2 - A3 - C2 - D3 - D4
Shade 3	A3.5 - A4 - B3 - B4 - C3 - C4

*Shade 1 = light shades - Shade 2 = medium shades - Shades 3 = dark shades

Clinical procedures
Restoration of class I, II, III, IV, V and MOD cavities

- Select the appropriate shade with daylight.
- Prepare the tooth cavity in a conventional manner. The margins are prepared by slightly beveling the occlusal surfaces and producing slight flares on interproximal cavities.
- Isolate the operating field from salivary contamination (rubber dam).
- For classes II, III and IV, a matrix band is placed and secured by wedges; it should be loosened to allow full contours when packing the restorative material into the cavity.
- If the cavity is closed to the pulp, apply a eugenol-free cavity liner (ex: calcium hydroxide, antibacterial liner).
- Etch the enamel and dentin.
- Apply the light curing bonding agent of your choice, following manufacturer's instructions.
- Apply Pureshade
- Polymerize* each layer. Refer to the table below for recommended curing times.
- Finish and polish the restoration.

*PURESHADE is intended to be cured by halogen or LED light with minimum light intensity of 650mW/cm2 in the 400-520nm range. Please refer to the table below for curing recommendation. Hold the light guide tip as close to the restorative as possible during light exposure. All times refer to a single occlusal surface cure. In any posterior restoration, additional 10 second cures on both the buccal and lingual surfaces of the tooth.

CURING TIMES	Shade 1 & 2	Shade 3 & Bleach
Incremented depth	2 mm	2 mm
Polymerization light with output 650 - 1200 mW/cm²	20s	40s
Polymerization LED light with output > 1200 mW/cm²	10s	20s

Note
The device is used on young adults to elderly people

Storage & shelf-life

- The product must be used by the expiration date indicated on the package.
- Do not expose the product to sun-light or warm sources.
- Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it should be brought back to room temperature 15 minutes minimum before use, in order to restore its normal viscosity and curing time.
- Use the product at room temperature.

Important notice

- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

Disclaimer

Dispose of the product in accordance with applicable local, regional and national regulations. This material is intended exclusively for use in dentistry. Processing must be done strictly in accordance with the instructions for use. No liability can be accepted for damage caused by failure to comply with the instructions for use or the prescribed area of application. It is the responsibility of the user to check the material for its suitability and use for a purpose not expressly stated in the instructions for use. Descriptions and information do not constitute a guarantee of properties and are not binding.

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