PUREFILL FLOW PUREFILL LIGHT FLOW

GB

Description of the products
Flowable light-cure composites.
Indications
All clinical cases where the use of a flowable composite is needed.
Specific indications
PUREFILL Flow
Splint for mobile teeth or after orthodontic treatment.
Little class III, class I and V
PUREFILL Light Flow
Cavity liner

- Restoration of small cavities
Pit and fissure
Safety notes
Warnings
- The products contain methacrylate resins which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
 - Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
 - Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
 - Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.
 - Avoid ingestion of these products. If signs of digestive disorder appear, contact a doctor.

- Precautions**
- Use of the products are restricted to dental personnel.
 - Products must not be used for any purpose other than specified in the indications.
 - Wear suitable protective eyewear, mask, clothing and gloves for dental professionals.
 - Protective eyewear is recommended for patients. Isolation of the operating field is requested (e.g. dental dam).
 - Dispose of these products according to the national and/or federal standards in force.

Contraindication
Products are contraindicated in patients with a history of allergic reaction to any of the components.
Important notice
Any serious incident that has occurred in relation to the products should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Incompatibilities
Products should not be used with eugenol-containing materials for pulp protection or temporary sealing since the eugenol can retard the curing process.

Composition
Dimethacrylate resins (35-45%) - Glass fillers (50-70%) - Campherquinone (<1%) - Accelerators (<1%) - BHT (<0.1%) - Pigments (<1%)
Clinical procedures
*PUREFILL Flow/Light Flow is intended to be cured by halogen or LED light with minimum light intensity of 650mW/cm2 in the 400-520 nm range. Please refer to the table below for curing recommendation. Hold the light guide tip as close to the restorative as possible during light exposure. All times refer to a single occlusal surface cure. In any posterior restoration, additional 10 second cures on both the buccal and lingual surfaces of the tooth.

Table 1

CURING TIMES	Light & medium shades	Dark, opaque & Bleach Shades
Incremented depth	2 mm	2 mm
Polymerization light with output 650 -1200 mW/cm²	20s	40s
Polymerization LED light with output > 1200 mW/cm²	10s	20s

Restoration of deficient enamel

1. Prepare the surface of the tooth by using a non-fluoride prophy-paste. Wash and dry.
2. Apply etching gel onto the enamel and dentine surfaces beginning with the enamel bevels. Rinse for 20 seconds with water. Dry it in with a water and oil-free air-spray, but do not desiccate. A slightly wet dentine surface is important for the function of last generation bonding agents. Properly etched tooth should have a dull, chalky-white appearance.
3. Isolation of the operating field is necessary (rubber dam) to avoid salivary contamination.
4. Select the desired VITA shade, keeping the tooth slightly moist.
5. Apply the light-curing bonding agent of your choice, according to manufacturer's IFU.
6. Apply PUREFILL Flow or PUREFILL Light Flow in layer of 2mm max. over the prepared enamel.
7. Polymerize* each layer. Refer to the table 1 for recommended curing times
8. Finish and polish the restoration.
9. On heavily discolored teeth (eg : tetracycline stains, fluorosis, radiological treatment, etc), the application of an opaque masking layer is recommended. In such cases, the restorative procedure is modified as follows: After cleaning, etching and bonding, as described above, apply a thin layer of the opaque as an under-layer and cure for 20 seconds. Apply the final coating of PUREFILL Flow or PUREFILL Light Flow over the opaque layer immediately after cure of the latter. For best adhesion, the opaque layer should be left unfinished, or its surface should be roughened.

Restorations of shallow Class I, class III and class V cavities

1. Prepare the cavity in a conservative manner, removing minimum tooth structure. Bevel the enamel adjacent to the exposed dentin.
2. Etch the dentin and enamel, as described in § 2 above.
3. Isolation of the operating field is necessary (rubber dam) to avoid salivary contamination.
4. Select the desired VITA shade, keeping the tooth slightly moist.
5. Apply the light-curing bonding agent of your choice, according to manufacturer's IFU.
6. Apply PUREFILL Flow or PUREFILL Light Flow in the prepared cavity.
7. Polymerize* each layer. Refer to the table 1 for recommended curing times
8. Finish and polish the restoration.

Sandwich technique (PUREFILL Flow only)

1. For MOD and class II, it is recommended to apply a layer of flowable composite prior to fill the cavity with a condensable composite, to get the best possible margin.
2. Once the tooth preparation, isolation and bonding application is done, apply a layer of PUREFILL Flow onto the surface of the cavity, in order to obtain a smooth surface allowing the condensable composite to reach every part of the cavity.
3. Polymerize* each layer. Refer to the table 1 for recommended curing times
4. Apply the condensable composite, following manufacturer's instruction for use.

PUREFILL Flow can also be used in conjunction with a bonding system, to secure mobile teeth with fiber glass.

Pits & fissures obturation (PUREFILL Light Flow only)

1. Prepare the surface of the tooth by prophylaxis, using a non-fluoride prophy-paste. Wash and dry.

2. Apply etching gel onto the enamel. Rinse for 20 seconds with water. Dry it in with a water and oil-free air. Properly etched tooth should have a dull, chalky-white appearance. Highly mineralized tooth may require an additional one to two minutes etching to achieve this effect.
3. Isolate the tooth to be restored from salivary contamination (rubber dam).
4. Apply a little quantity of PUREFILL Light Flow of the desired VITA shade at the dry surface of the fissure. Using a disposable brush (Fine or X-Fine size), push the flowable composite down to the bottom of the fissure.
5. Polymerize* each layer. Refer to the table 1 for recommended curing times
6. Finish and polish the restoration.

Products are used on young adults to elderly people.

Storage & shelf-life

- Products must be used by the expiration date indicated on the package.
- Do not expose the products to sun-light or warm sources.
- Store at a temperature between 5°C and 25°C. When the products are stored in the refrigerator, they should be brought back to room
- temperature 15 minutes minimum before use, in order to restore their normal viscosity and curing time.
- Use the products at room temperature.

Important notice

- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit their polymerization.
- Storage at a too low temperature leads to the decomposition of the products which will render it completely unusable.

Disclaimer

Disposal of the products must be carried out in accordance with local, regional and national regulations in force. These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions for Use or the stipulated area of application. The user is responsible for testing the materials for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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