## SEALTEMP S SEALTEMP S CLEAR

### Description of the product

Resin-based temporary luting cement. Well-balanced adhesive properties provide good prosthesis retention. Excellent resistance to oral environment and mechanical strength to cement good long-term performance. Low film thickness of cement.

*Versatile in use:*

- Cementation of provisory prosthodontic restorations on teeth and implant abutments.
- Cementation of definitive prosthodontic restorations on implant abutments.
- Cementation of definitive aesthetic elements for trying purpose.
- Cementation of provisory veneers.

#### Safety notes

#### Warnings

- Contains polymerizable methacrylate resins which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.
- Avoid ingestion of the product. If signs of digestive disorder appear, contact a doctor.

#### Precautions

- The use of this product is solely restricted to dental personnel.
- The product must not be used for any purpose other than specified in the indications.
- Dental professional should wear suitable protective eyewear, mask, clothing and gloves.
- Protective eyewear is recommended for patients.
- Dispose according to the national and/or ederal standards in force.

#### Contraindication

Product is contraindicated in patients with a history of allergic reaction to methacrylate resins and BHT or to any of the components.

#### Important notice

- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

#### Incompatibility

- Seal Temp should not be used with eugenol-containing materials for pulp protection or temporary sealing, since the eugenol can retard the curing process.
- Seal Temp should not be used with hemostatic containing ferric compounds since these materials may impair adhesion and may cause discoloration of the tooth margin or surrounding gingiva due to remaining ferric ions.
- Seal Temp cannot be used with a hydrogen peroxide solution for cleaning cavities since it may weaken the bond strength to the tooth structure.

#### Composition

Dimethacrylate resins (80-90%) - Glass fillers (1-10%) - Accelerators (<1%) - BHT (<1%) - Pigments (<1%).

#### Times

Total time (working time + setting time) at 37°C, is 1'20" max

#### Clinical procedures

*First use of a new automix-syringe*

- Remove and discard the cap of the automix-syringe. Under no circumstances should the cap of the automix-syringe be used to reseal the automix-syringe as this could result in the inclusion of air bubbles, inducing a cross-contamination (base/ catalyst) and leading to a prematured polymerization of the product.
- Check whether both automix-syringe openings are free and remove obstructing material, if any, with a suitable instrument.
- Before installing a mixing tip, bleed the automix-syringe onto a mixing pad until the base and catalyst pastes are at the same level.

#### Seal Temp S, automix-syringe form

- Remove automix-syringe cap, dispense and discard a small amount of material. Be sure the material is flowing freely from both ports.
- Install a mixing tip on the automix-syringe.
- Gently depress syringe plungers, to begin the flow of material. Do not use excessive force. Extrude enough material to make a thin layer of cement inside the temporary crown.
- Hold the temporary crown firmly in place.
- Trim excesses immediately.
- Keep the automix-syringe with the mixing tip in place until the next use.

#### Seal Temp S, 2 syringes form

- Dispense approximately equal amount of Part A and Part B pastes onto a mixing pad.
- Mix the pastes thoroughly with a spatula for 10 to 15 seconds
- Spread a thin layer of cement inside the temporary crown and hold the crown firmly in place.
- Trim excesses immediately.

#### Notice

Seal-Temp S is retentive enough for normal height abutment teeth.

This device is used on young adults to elderly people, since definitive prosthodontic sessions are only conducted on adults.

#### Storage & shelf-life

- The product must be used by the expiration date indicated on the package.
- Do not expose the product to sun-light or warm sources.
- Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it should be brought back to room temperature 15 minutes minimum before use, in order to restore its normal viscosity and curing time.
- Use the product at room temperature.

#### Important notice

- Leave the used mixing tip in place on the automix-syringe until next use.
- Do not throw away the box and the instruction for use until the product is finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.

#### Disclaimer

Disposal must be carried out in accordance with local, regional and national regulations in force.

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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